



1) Pair Bracelet

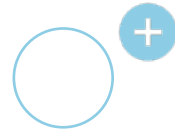

For patient if you pair the Corsano Bracelet for first time to your mobile device

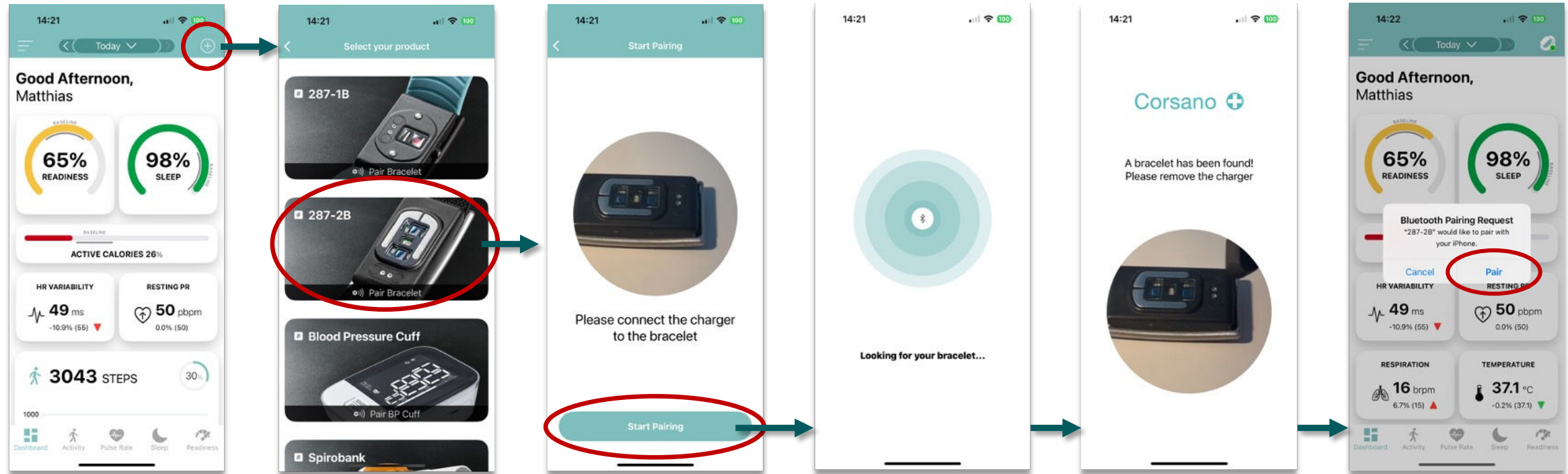

1) Press  right top corner to go to Settings


2) Select bracelet

3) Press [Start Pairing]

4) Wait for the end of the scanning process


5) Remove the charger from the bracelet to confirm the pairing


6) Secured bonding confirmation. Press [Pair]

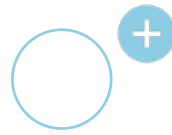


2) Unpair Bracelet


For patient if you previously paired a Corsano Bracelet to your mobile device

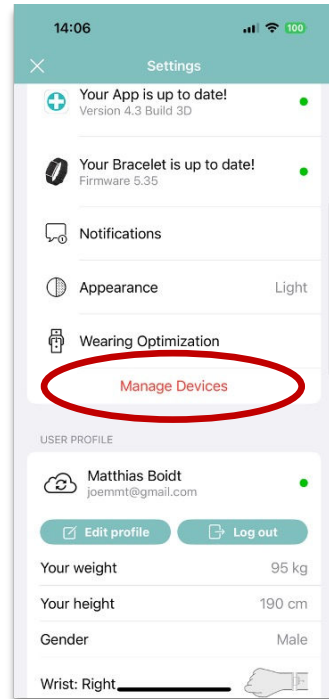


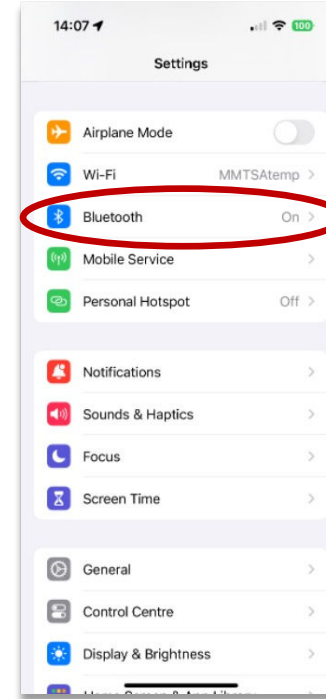
1) Press  at top corner to go to Settings and select [Manage Devices]

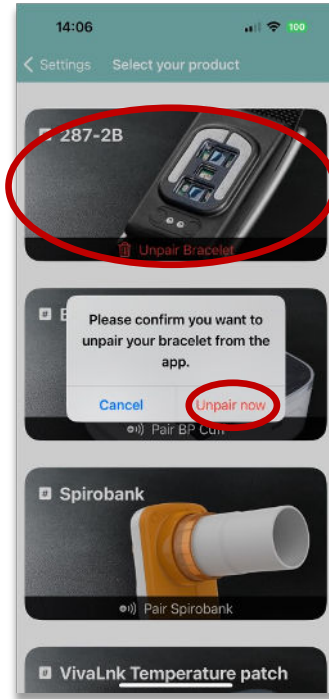


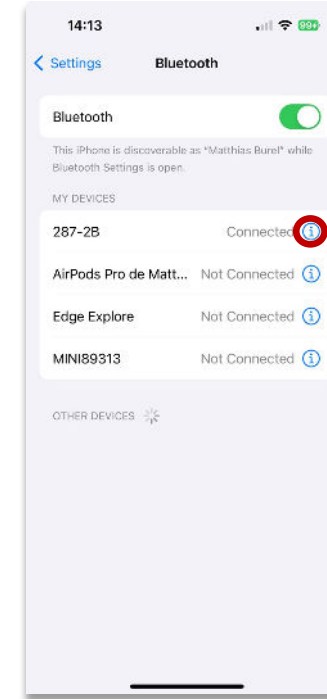
2) Select bracelet and confirm when you receive the pop-up

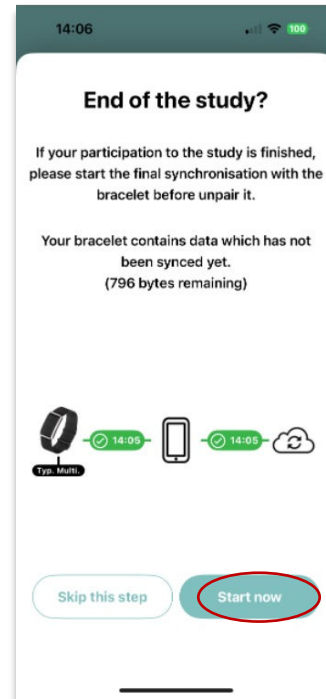


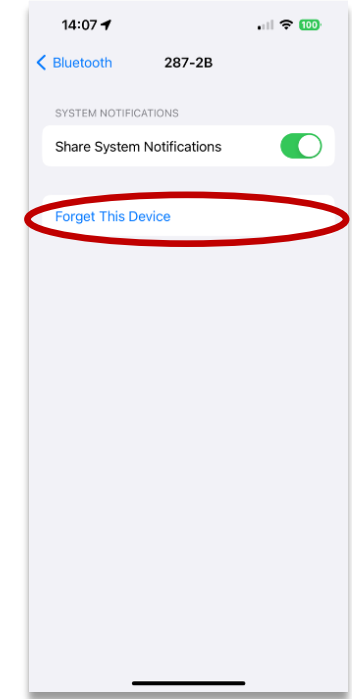
3) Press [Start now] and you will see 
Wait until data is synced



4) You must remove the bracelet from mobile Bluetooth Settings. Go to 



5) Select the 287-2B device in the iOS Bluetooth peripheral list. Click on the  icon



6) Click on [Forget This Device] to remove the secured bonding