



1) Registration with Email

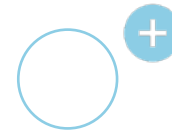

Download the Corsano app and create an account

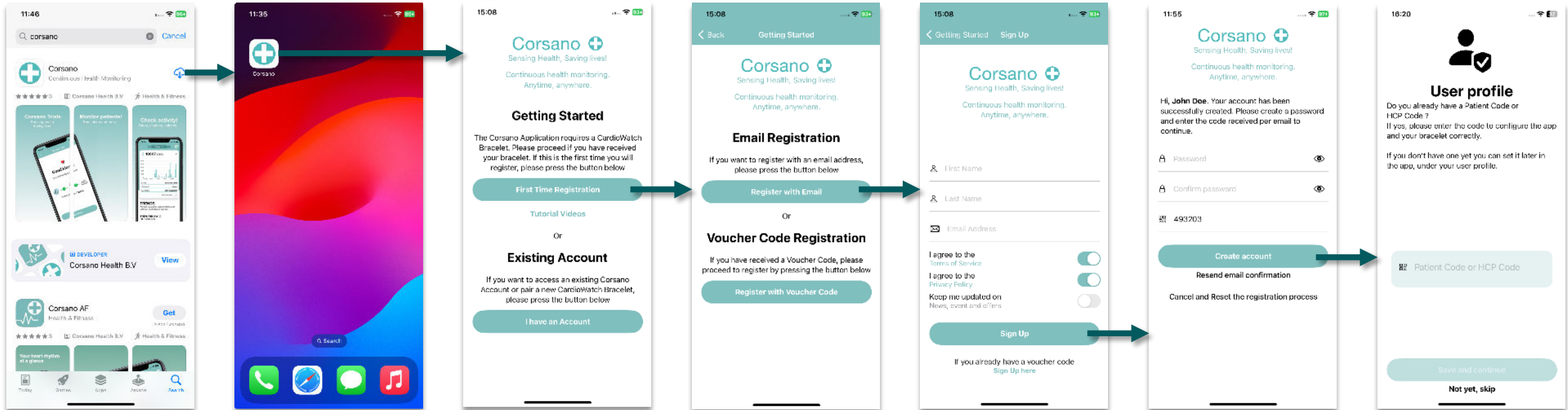

1) Search for Corsano on App Store or use following link

2) Install App and start from home screen

3) Press on [First Time Registration]

4) Select the option [Register with Email]

5) Type First Name, Last name and email
Check your email account to get the verification code

6) Set a password and the verification code received by email to complete the account creation


7) Enter the patient code

1) Register Gateways in Study



Add a Gateway to your study

1 – Select Gateway

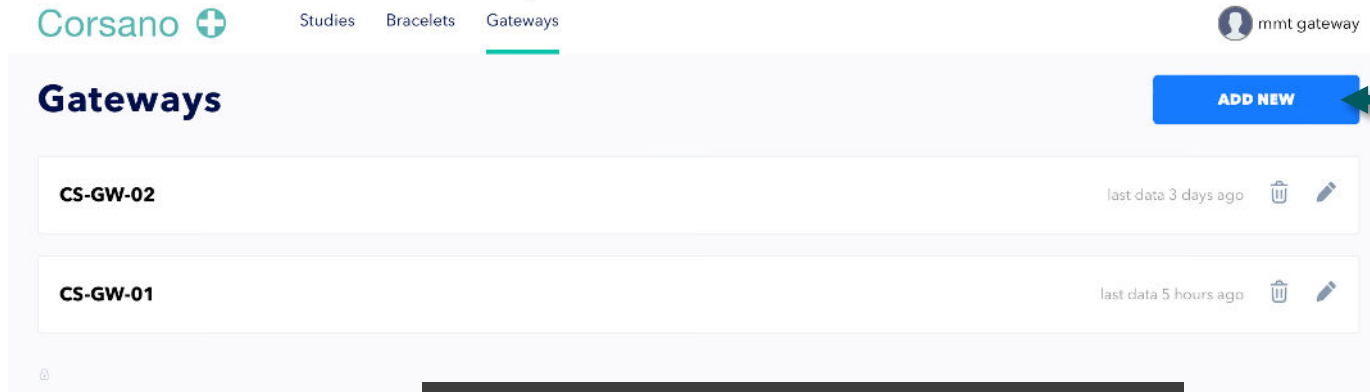

2 – Click Add New



New Gateway X

Name


Serial number

SAVE

3 – Set a name and the serial number linked to the Gateway



2) Register Bracelets in Study Portal

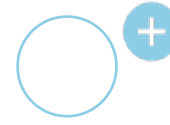

Add a Bracelet to your study

1 – Select Bracelets



Corsano

Studies **Bracelets** Gateways

mmt gateway

Bracelets

ADD NEW

10000523 287-2 F2:95:07:33:C0:05	last data 5 hours ago
10000524 287-2 D8:93:BF:E2:AB:1C	last data 2 days ago
10000525 287-2 F2:79:A7:90:DA:C0	last data 5 hours ago

2 – Click Add New



New Bracelet

Name

Bracelet Type

287-1

Mac address

SAVE

3 – Set a name, select the bracelet model and set the Mac address. The Mac address will be provided by Corsano



3) Register Patients in Study Portal



Add a Patient to your study

1 – Select Vouchers



Study / mmt pilot22 [TJ6PU]

Vouchers Export

Patient	BPM	O ₂ Saturation	BRPM	Steps	Temperature
Patient BAPGKTSM 10000523	75	98	12.8	3151	36.9
Patient 9MC22UAD 10000524	73	97	11.8	2265	37.8
Patient 5 HCAHJ15B 10000525	79	100	15.5	30	37.0
Patient 72PFGZZT Corsano trials	65	80	13.3	3656	38.1

2 – Select the number of vouchers you want to add and press “Add users”



New voucher Export vouchers


Number of new vouchers you want to create.

1 Add users

3 – You will see the vouchers appearing in this list

4) Setup the Gateway in Corsano app


Configure the Gateway with Corsano app

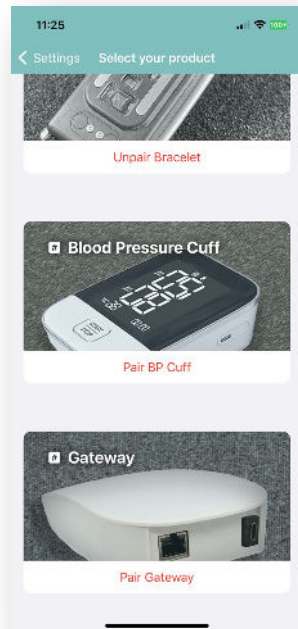


1) Select the Gateway product in:
Main menu -> Manage Devices




2) Press on Start Pairing

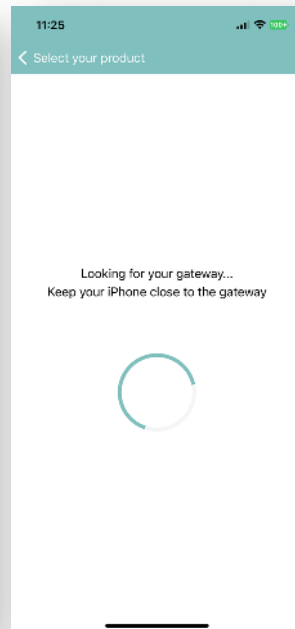


3) The app is looking for Gateway peripherals around you

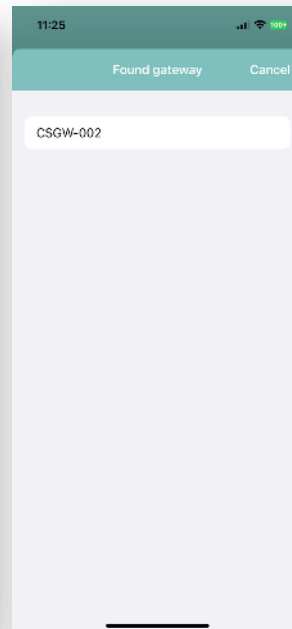


4) The app display a found Gateway list. Select the one you want to setup

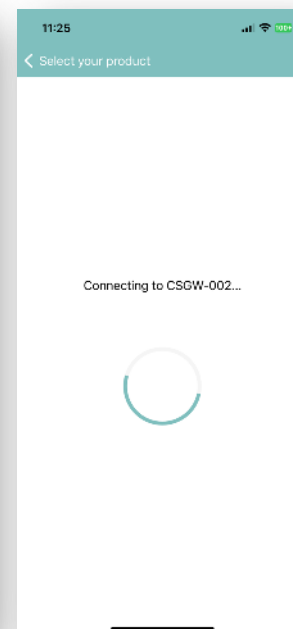

5) Connecting process. The app will ask for pairing, please accept.

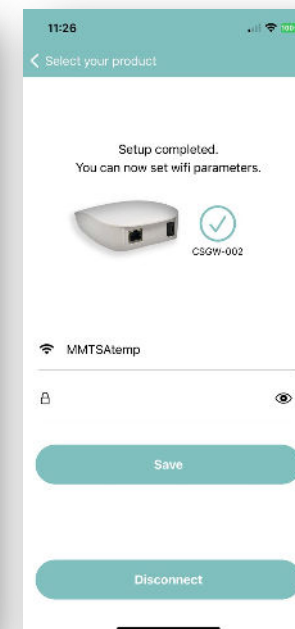

6) Once the connection is ok, enter your WIFI ssid name and password.

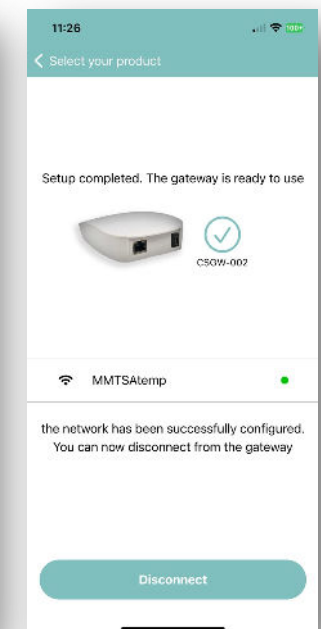

7) The app and Gateway are checking the internet connection. If ok, the app display a green marker. If not, a red marker.

Finalize by pressing on Disconnect

Remove the Gateway from iOS bluetooth peripheral list when the setup is done.
Go to iOS general settings-> Bluetooth
Select the Gateway and press on « i » icon, then Forget this device