



1) Access to the portal

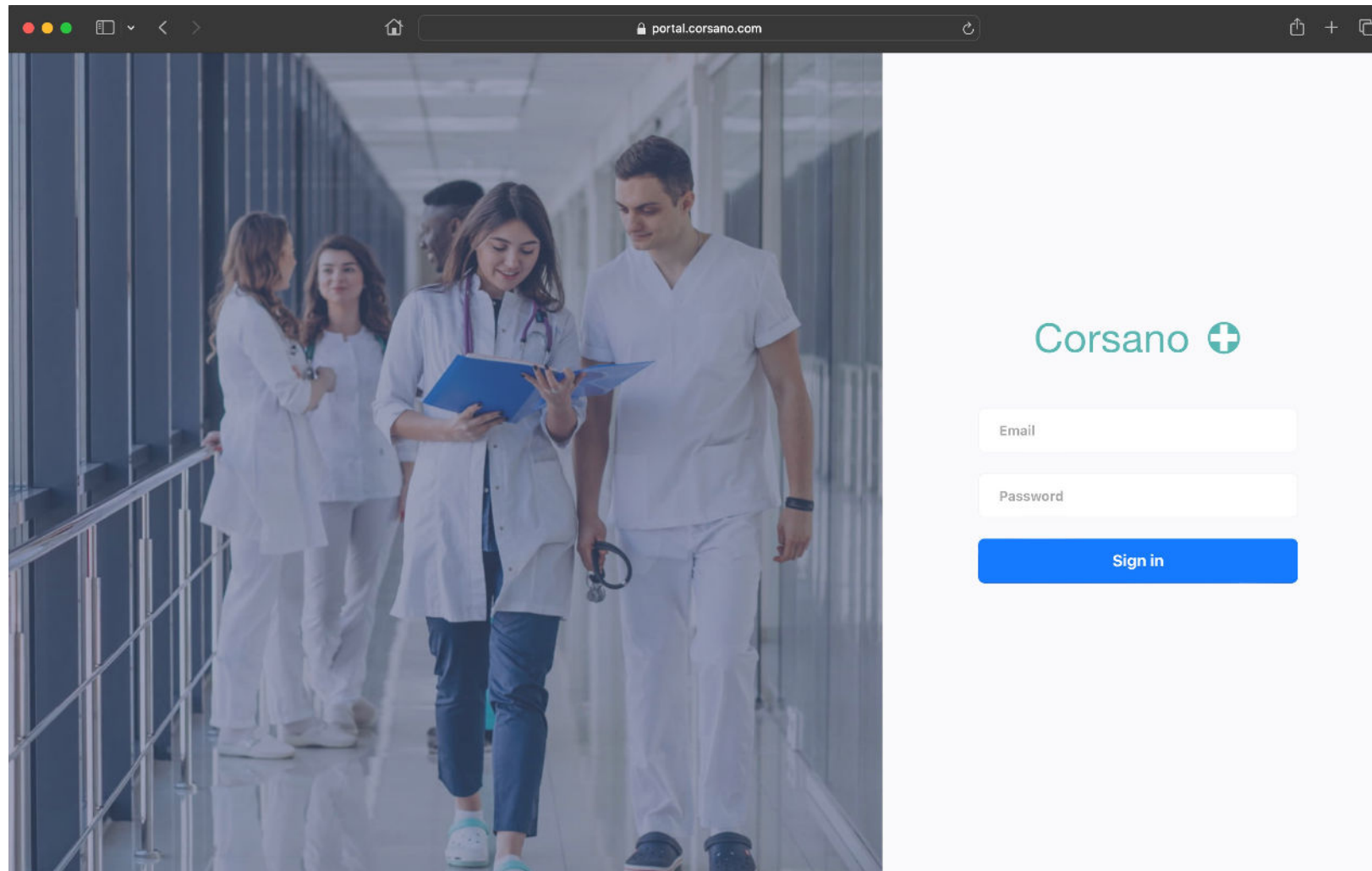


Go to <https://portal.corsano.com> and sign in with your credentials
(Create your account within the corsano application)


2) Departments list

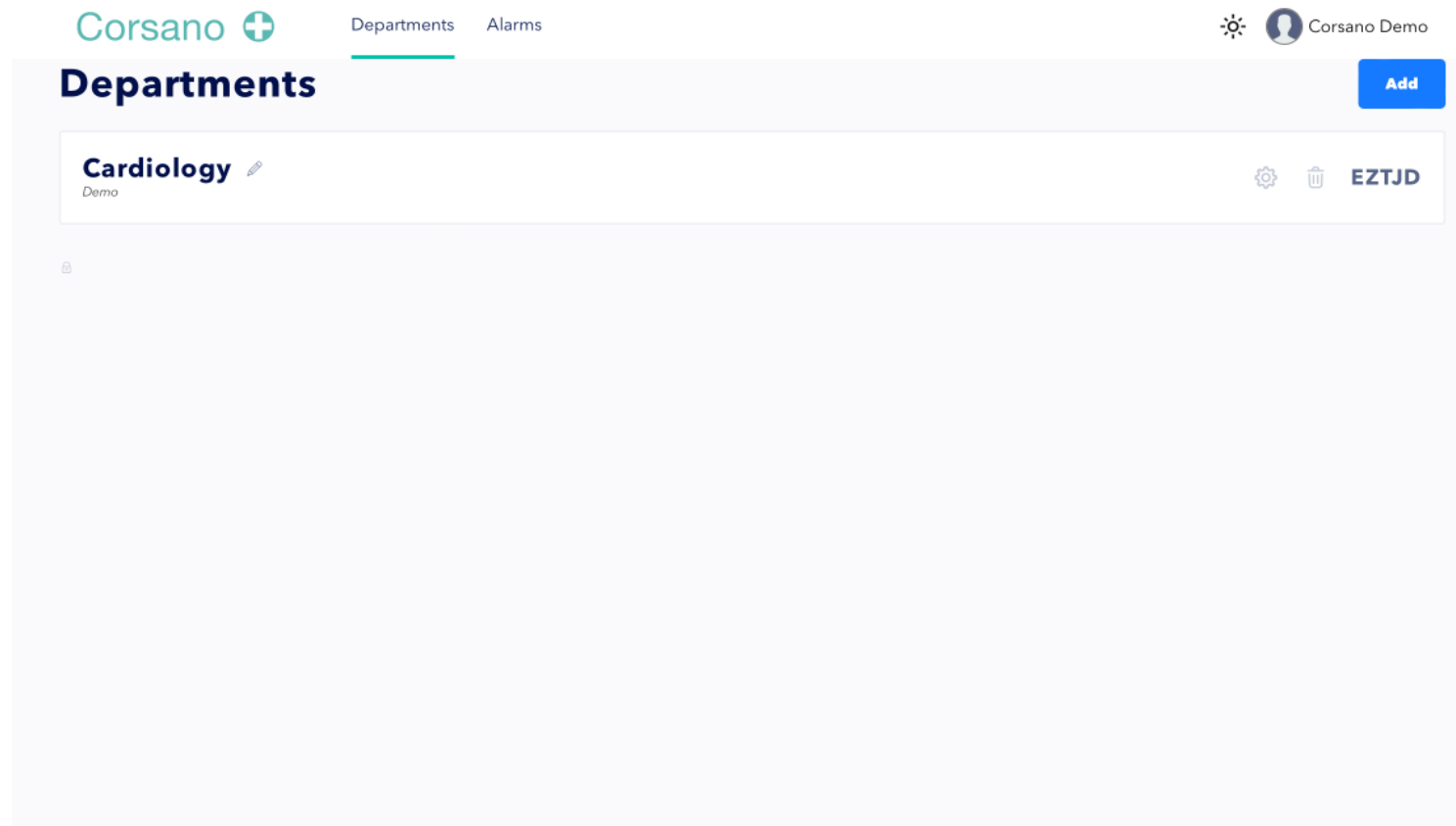





Once authenticated, you can get access to all your departments

3) Patient details


Corsano  Departments Alarms ☀️  Corsano Demo

 **Matthias Boidt** [EDIT PROFILE](#)

 **15:56:06, Apr 2**
a minute ago  1984 (40 y.o.)  95 kg  190 cm  male


Compliance ⓘ < March 2024 >

1 Fri 99.0%	2 Sat 99.7%	3 Sun 99.7%	4 Mon 89.2%	5 Tue 100.0%	6 Wed 99.7%	7 Thu 99.4%	8 Fri 96.8%	9 Sat 99.4%	10 Sun 99.3%	11 Mon 100.0%	12 Tue 99.3%	13 Wed 94.1%	14 Thu 99.7%	15 Fri 93.5%	16 Sat 99.7%
17 Sun 100.0%	18 Mon 93.8%	19 Tue 99.7%	20 Wed 100.0%	21 Thu 99.0%	22 Fri 93.4%	23 Sat 99.3%	24 Sun 99.3%	25 Mon 94.4%	26 Tue 100.0%	27 Wed 99.7%	28 Thu 100.0%	29 Fri 97.9%	30 Sat 97.9%		

Statistics <  April 2 >

Overview Pulse Rate SPO2 Respiration Activity Temperature BP Spiro Weight

Pulse Rate

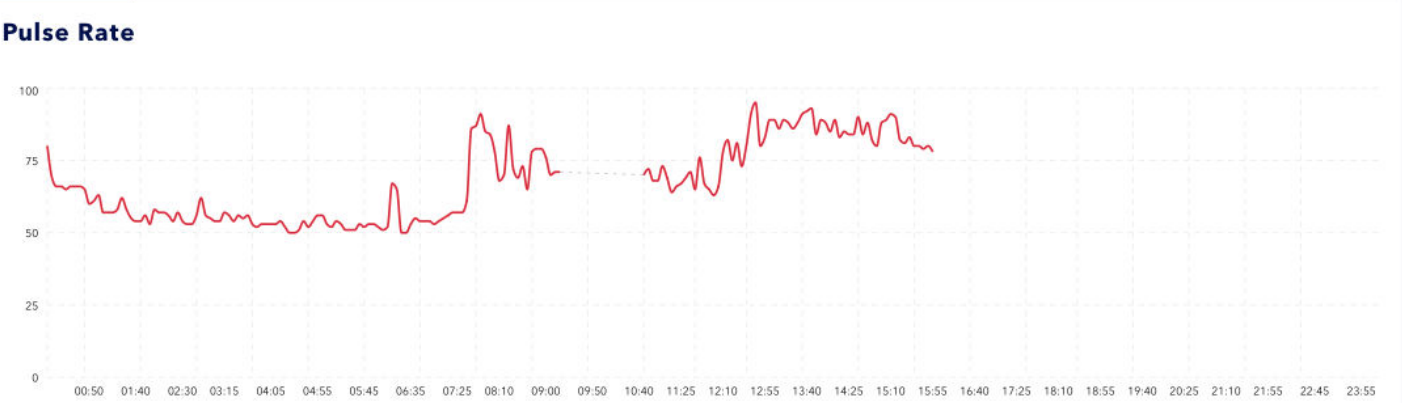

00:50 01:40 02:30 03:15 04:05 04:55 05:45 06:35 07:25 08:10 09:00 09:50 10:40 11:25 12:10 12:55 13:40 14:25 15:10 15:55 16:40 17:25 18:10 18:55 19:40 20:25 21:10 21:55 22:45 23:55

Select a department to see the patient list. Then click on a patient to see details

4) Alarms details

Event Type	Date	Time	Patient	Vital	Value	Limit	Priority	Status
Alarm condition	2023-11-01	10:27:12	Daniel Jean	Pulse Rate	125	117	Medium	Audio on
Alarm condition	2023-11-01	10:32:46	Daniel Jean	Pulse Rate	145	139	High	Audio on
Alarm condition	2023-11-01	10:37:33	Daniel Jean	SpO2	87	90	Medium	Audio on
Alarm condition	2023-11-01	10:41:01	Daniel Jean	SpO2	80	85	High	Audio on
Alarm condition	2023-11-01	10:43:07	Daniel Jean	Respiration Rate	25	20	Medium	Audio on
Alarm condition	2023-11-01	10:49:30	Daniel Jean	Respiration Rate	35	30	High	Audio on
Alarm condition	2023-11-01	10:54:17	Daniel Jean	Temperature	98	96	Medium	Audio on
Alarm condition	2023-11-01	11:01:48	Daniel Jean	Temperature	98	97	High	Audio on
Alarm inactivation	2023-11-01	11:05:10	Daniel Jean	Temperature	98	80	High	Pause Audio
Alarm inactivation	2023-11-01	11:08:29	All	N/A	N/A	N/A	N/A	Pause All Audio