



# 1) Start Monitoring New Patient




If you want to start monitoring a new patient with same app and same bracelet.

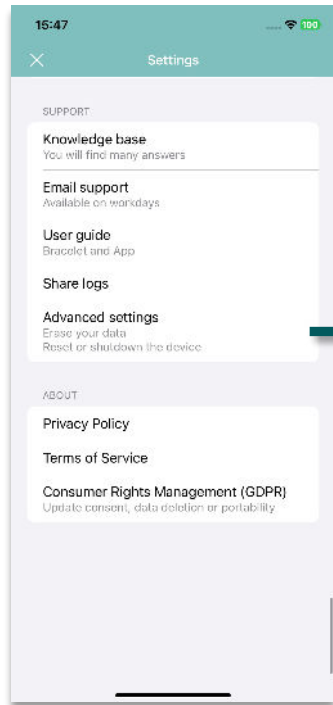


1) Open "Advanced settings" in app main menu

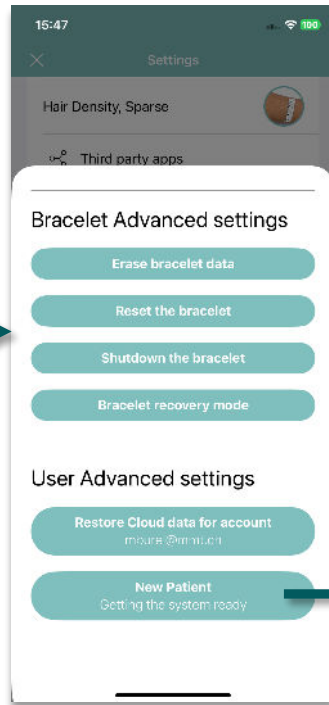


2) Select "New Patient"

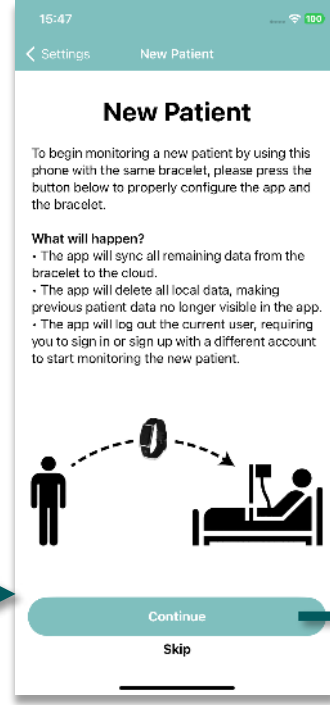


3) Read instructions and continue

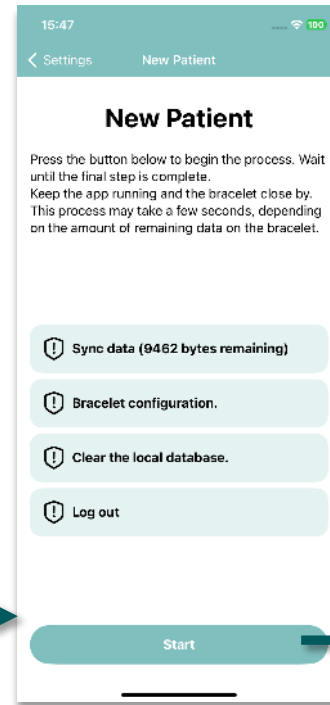

4) Read instructions and start the process

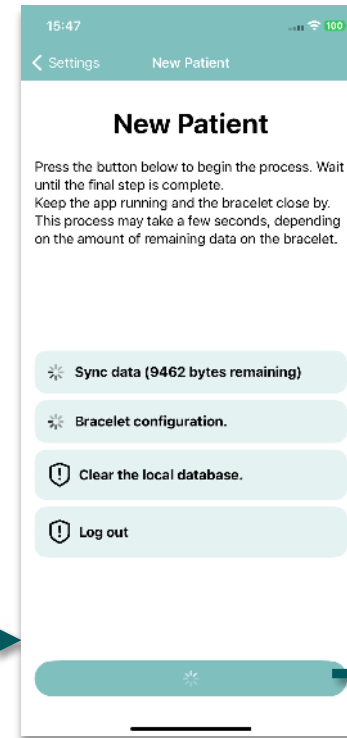

5) Process is running

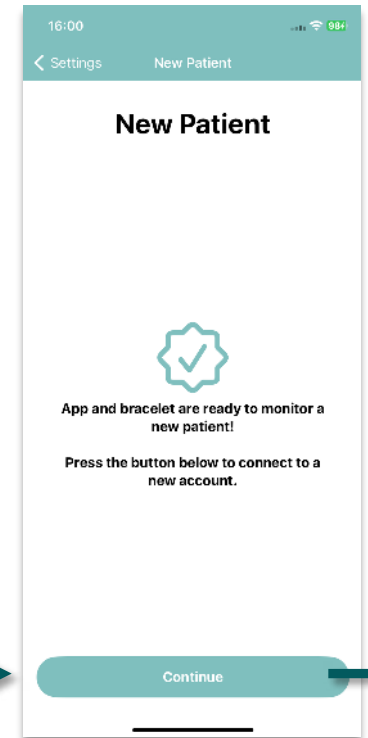


6) Press "Continue" to register a new patient

# 2) Start Monitoring New Patient



Register a New Patient with a voucher code

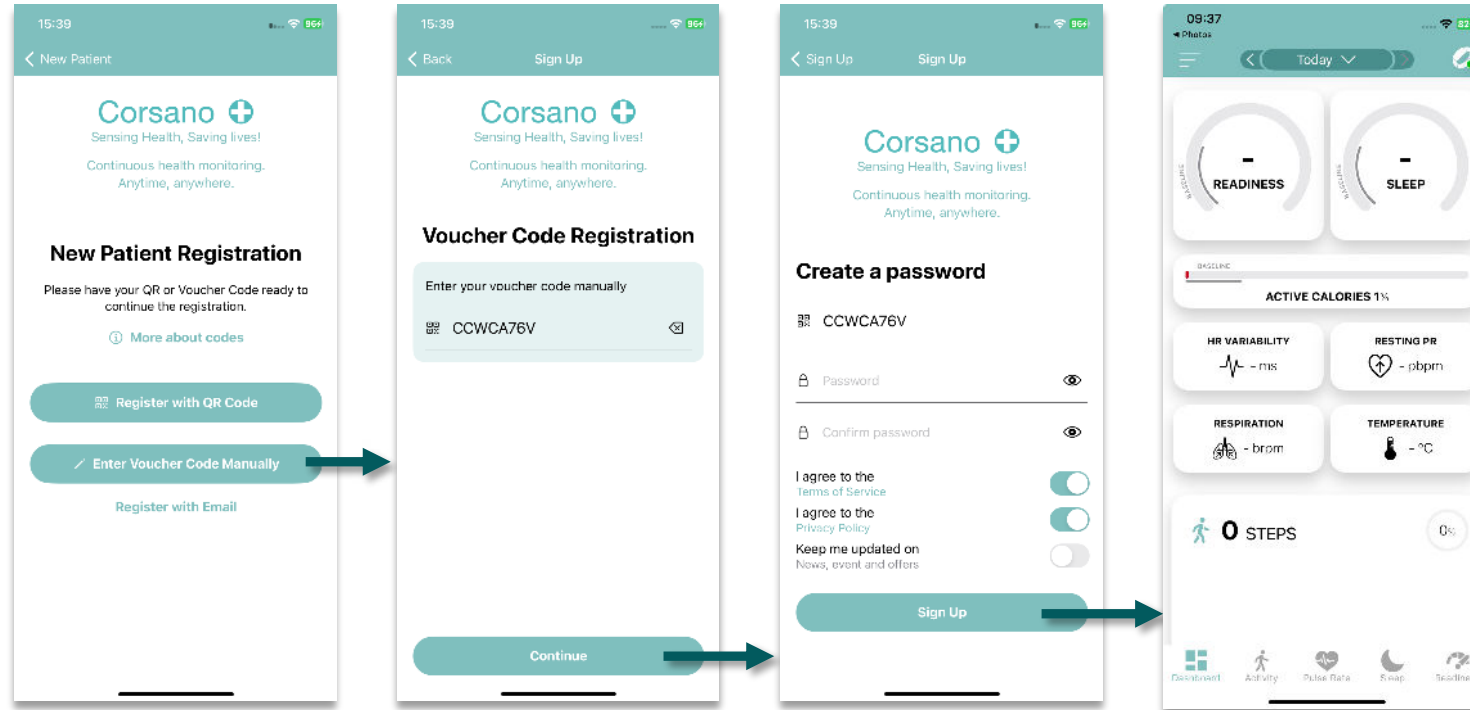


1) Select if you want to enter the Voucher Code manually or by scanning the QR Code

2) Enter the Voucher Code


3) Set a password and press "Sign Up" to finish the Patient registration.

5) Ready to monitor new patient


# 1) Start Monitoring New Patient

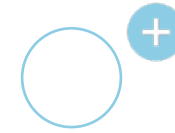

If you want to start monitoring a new patient with same app and same bracelet.

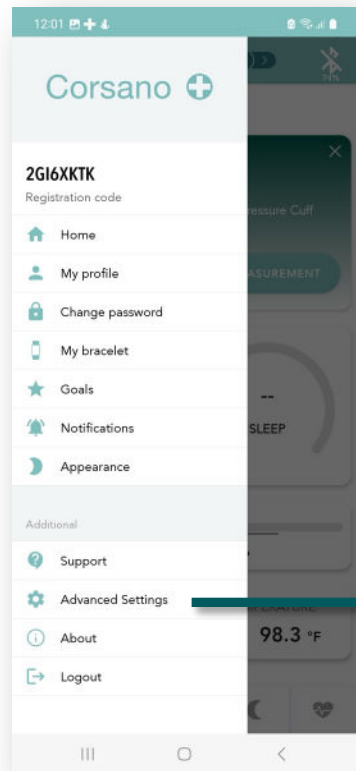


1) Open "Advanced settings" in app main menu

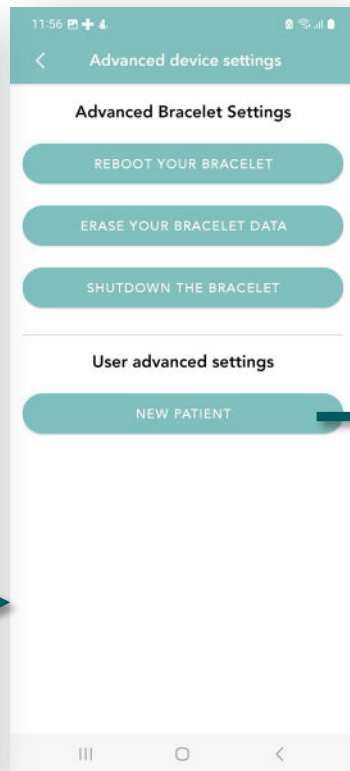


2) Select "New Patient"




3) Read instructions and continue

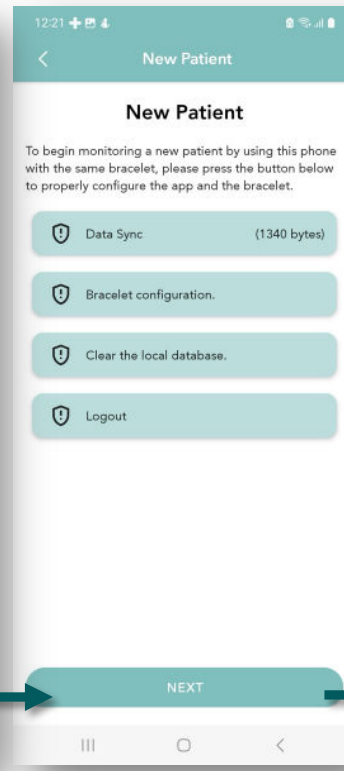

4) Read instructions and start the process

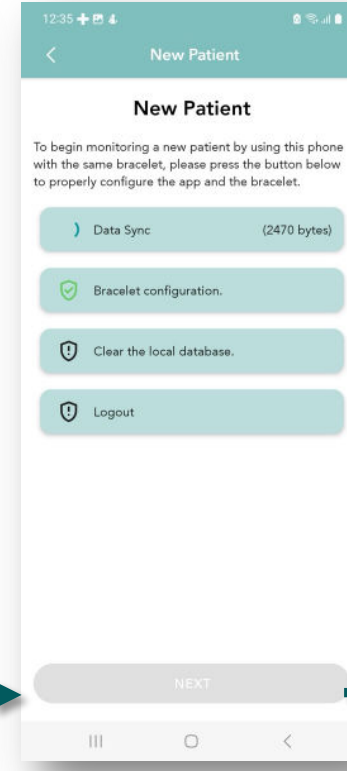


5) Process is running

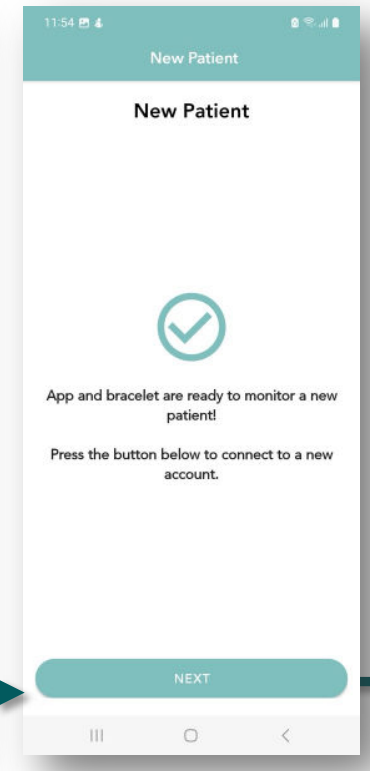

6) Press "Continue" to register a new patient

# 2) Start Monitoring New Patient

Register a New Patient with a voucher code

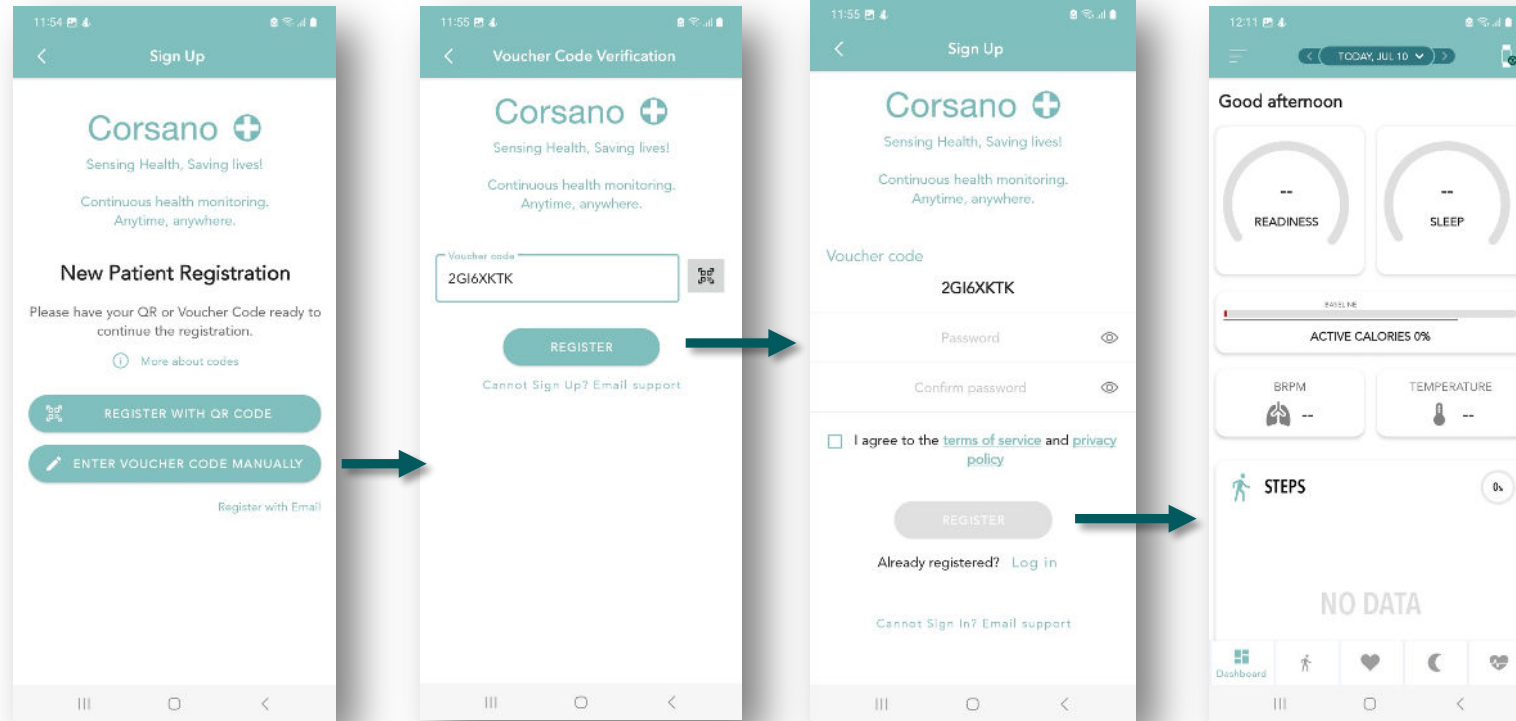

**1) Select if you want to enter the Voucher Code manually or by scanning the QR Code**

**2) Enter the Voucher Code**

**3) Set a password and press "Sign Up" to finish the Patient registration.**

**5) Ready to monitor new patient**