


1) Registration with Email 1/2




Go to <https://portal.corsano.com/>

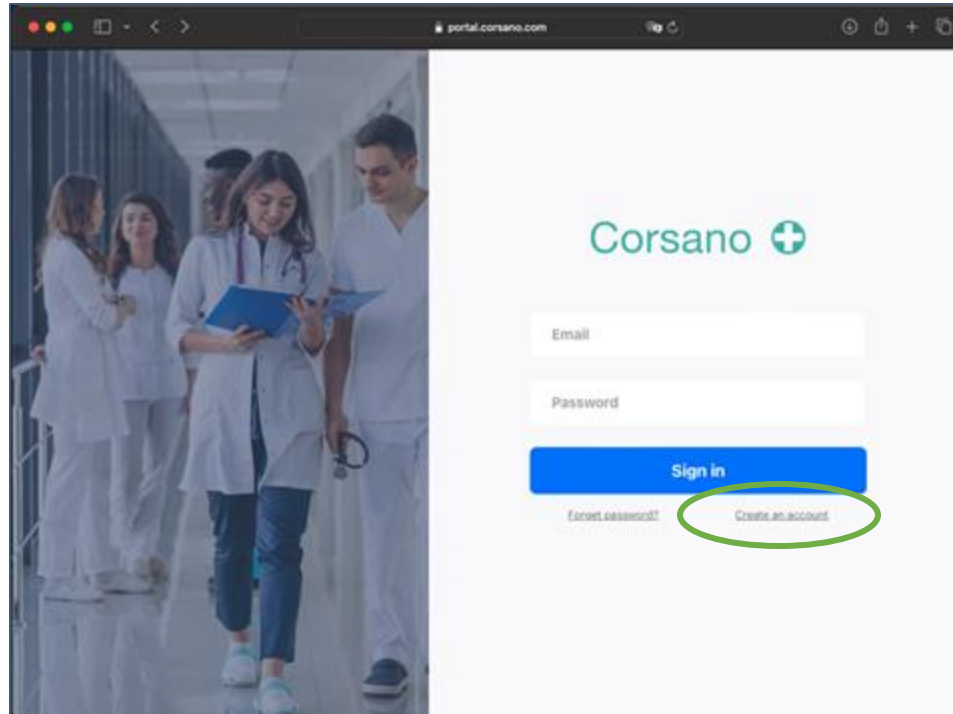

1) Go to <https://portal.corsano.com/>

Click on "Create an account"


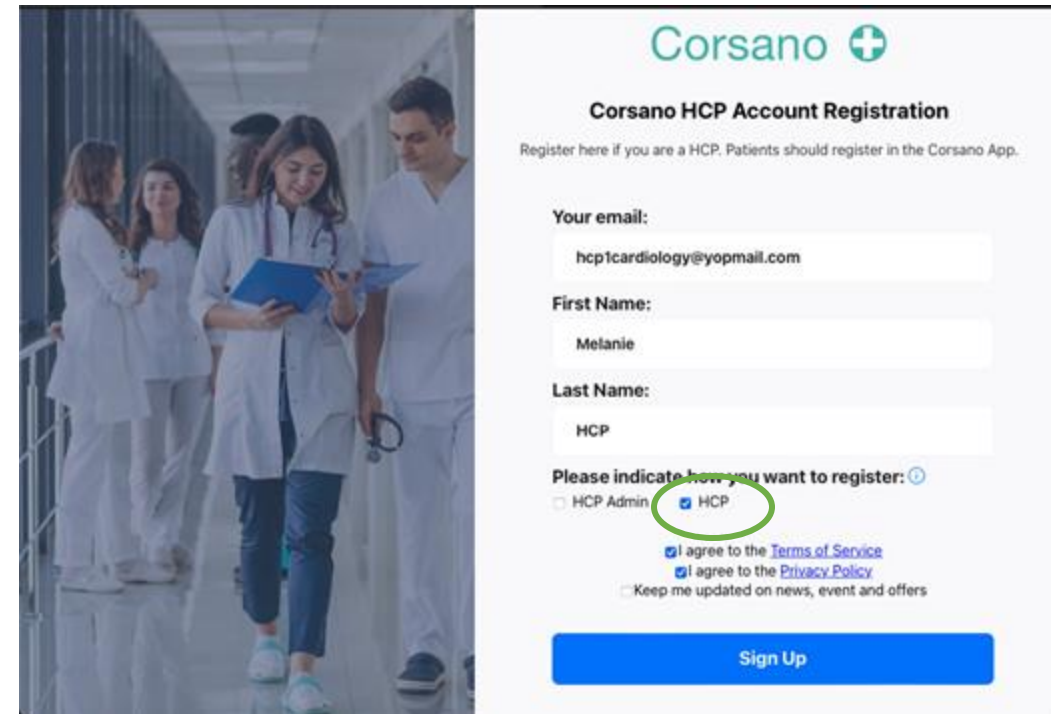


2) Type First Name, Last name and email

Select HCP


Check your email account to get the verification code

1) Registration with Email 2/2

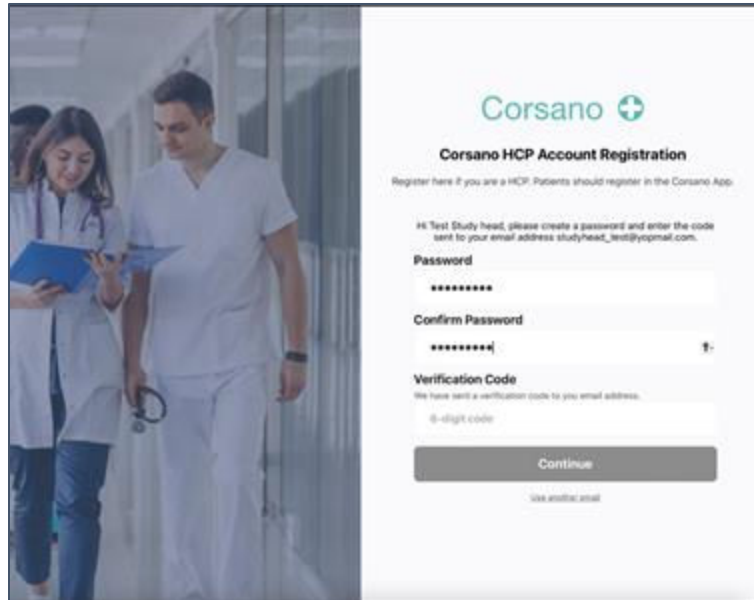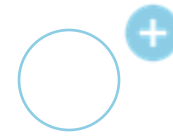


1) Set password

Check your email account to get the verification code

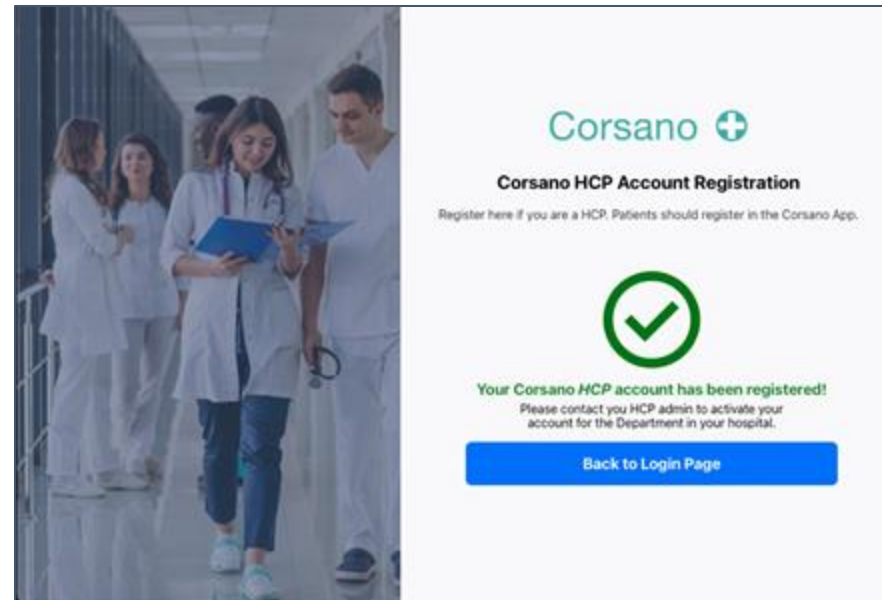


2) Account successfully created

Login with your credentials

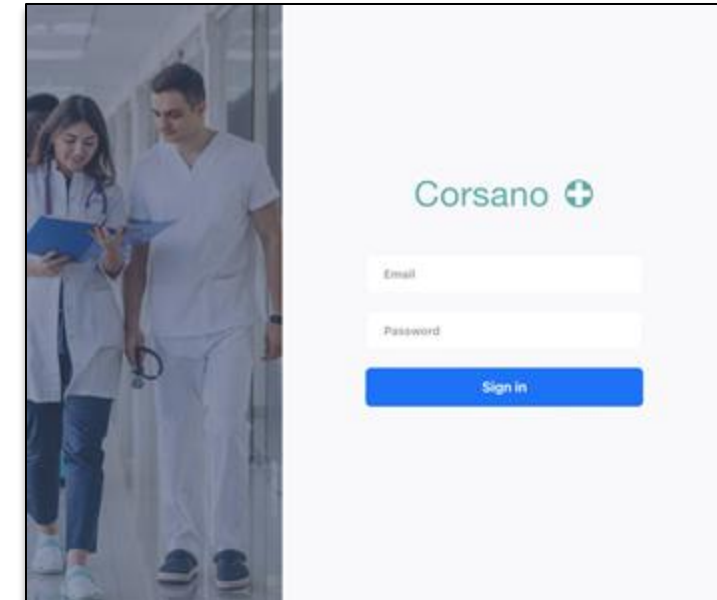


2) Access a Study/Department

After login to the Portal

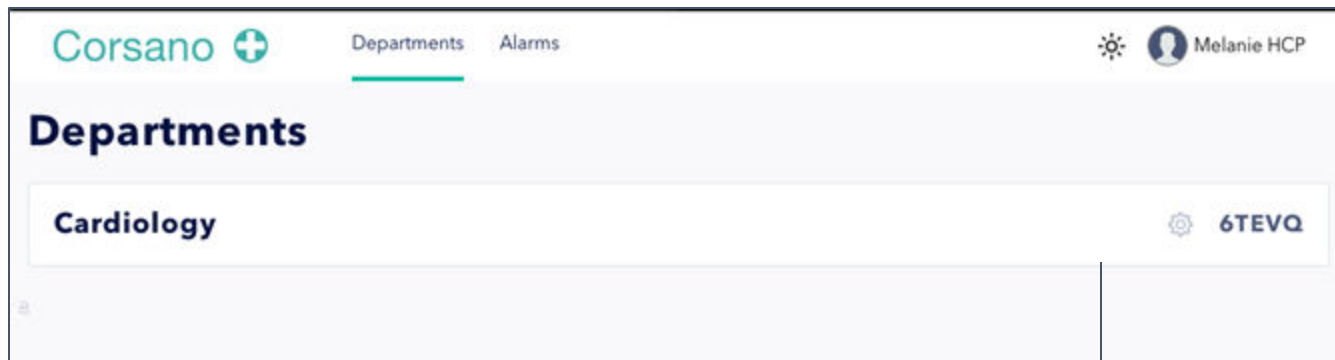


A study should appear

If not, please contact your HCP admin to be given access

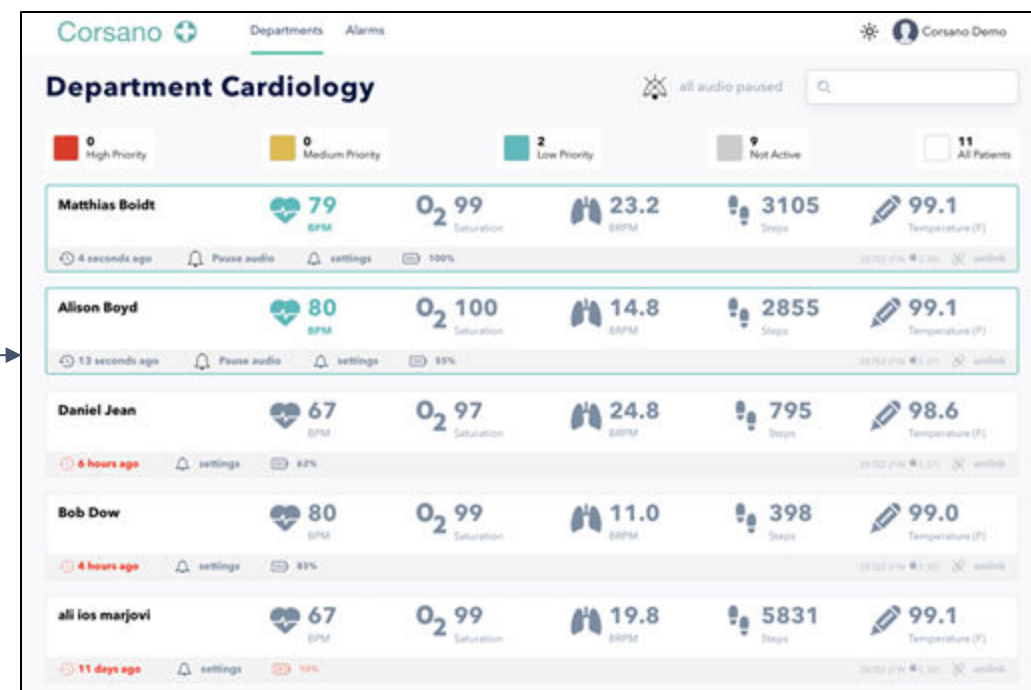

3) Create Vouchers (optional)


Create anonymous vouchers for your patients

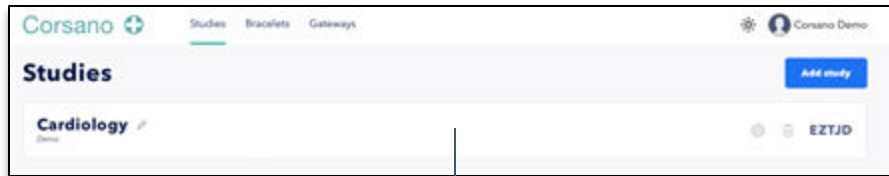


1) Select the Study you want to create vouchers for



2) Click on [Vouchers]

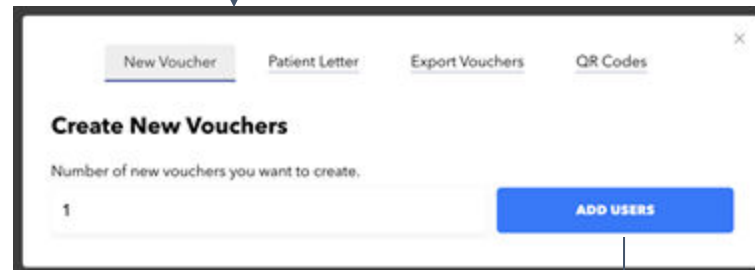


3) Enter required Study Participant number in the dialogue box. Click [Add users]

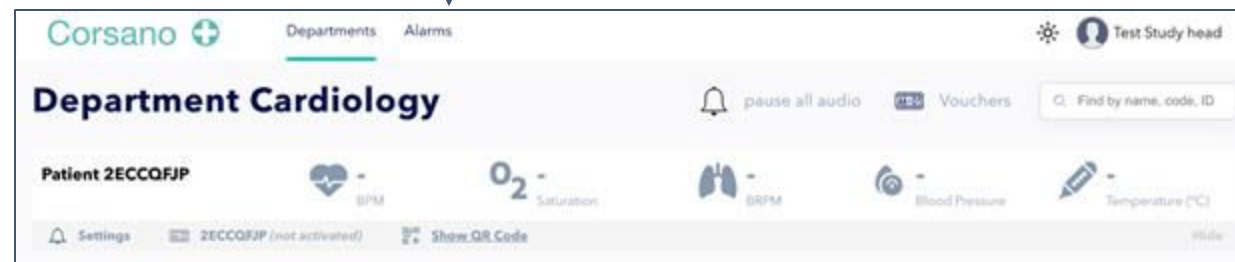


4) Participant created

4) Share Vouchers (optional)

Option to export vouchers and personalized invitation letters for your patients

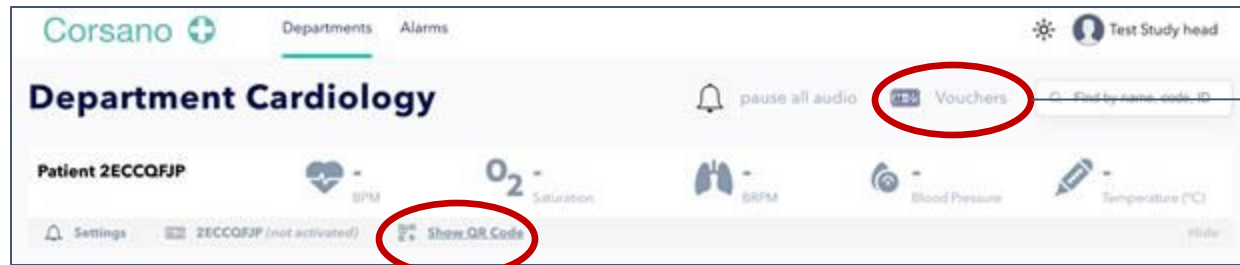

Export all

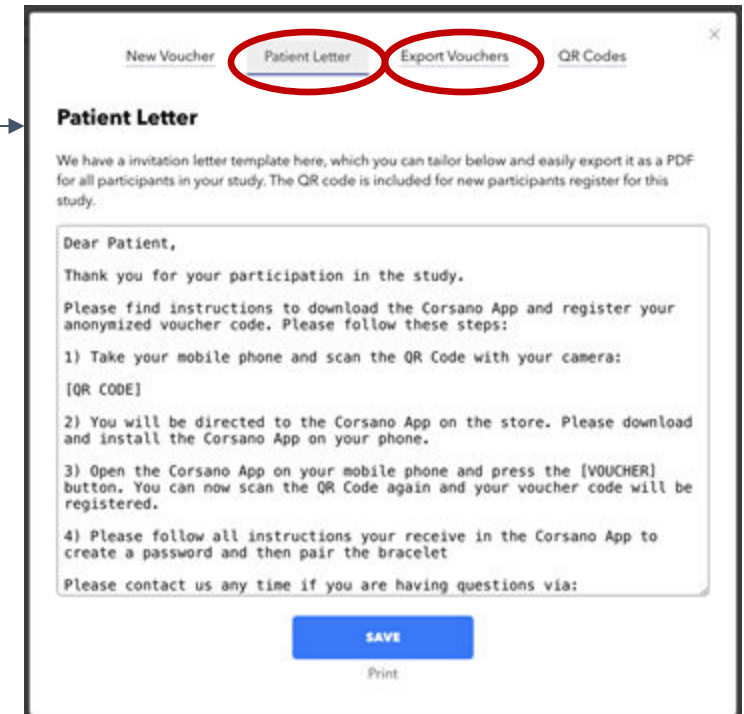



QR code for selected patient










Click on [Patient Letter] to export all letters including QR Code for patients.





Or click on [Export vouchers] to get an XLS file of vouchers

5) Alarms threshold (optional)

Corsano  Departments Alarms ☀️  Test Study head

Department Cardiology  pause all audio  Vouchers

Patient 2ECCQFJP  - BPM  - Saturation  - BRPM  - Blood Pressure  - Temperature (°C)

 Settings  2ECCQFJP (not activated)  [Show QR Code](#)  Hide

Alarms 

Pulse Rate

 36 50 91 111 131

Spo2

 90 93 96

Temperature

 35 36 38 39

Set default values

Save