



1) Download Corsano App with QR Code


For patient if you have received an invitation letter with QR Code

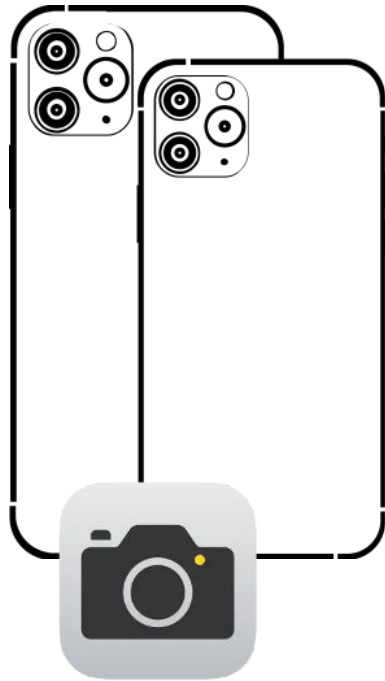


1) Open the Camera App on your mobile

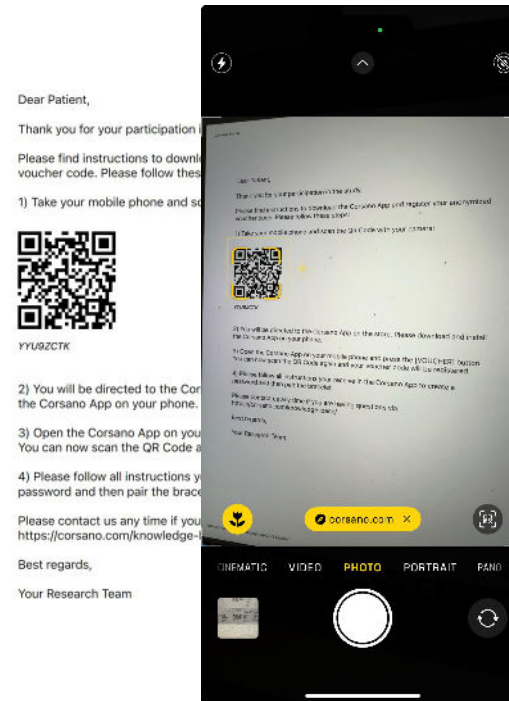

2) Take your letter and scan the QR-Code

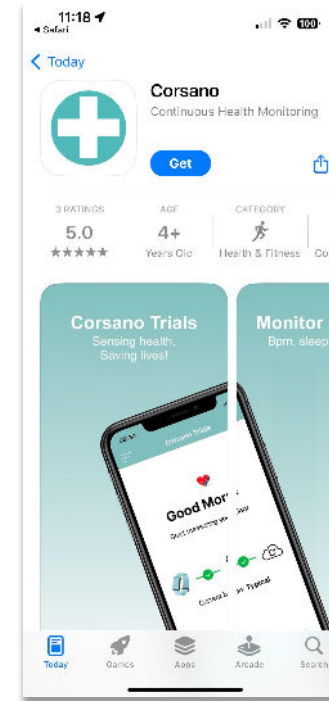


3) The App Store will automatically open. Click to install Corsano App

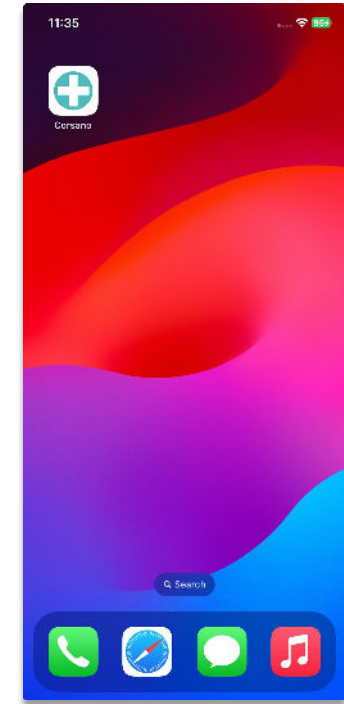


4) App ready to use


2) Registration with QR Code

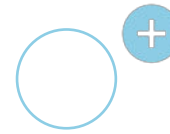


For patient if you have received an invitation letter with QR Code

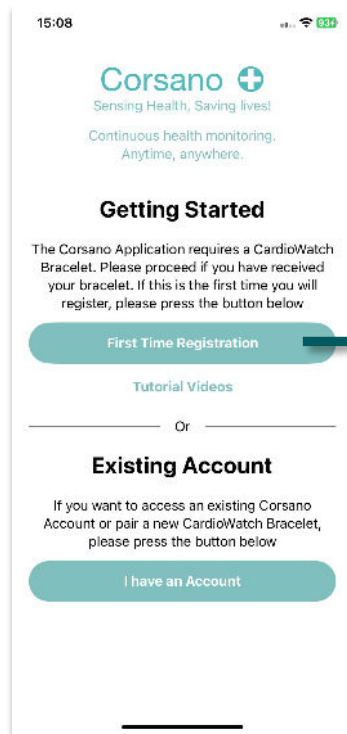


1) Click on [First Time Registration]

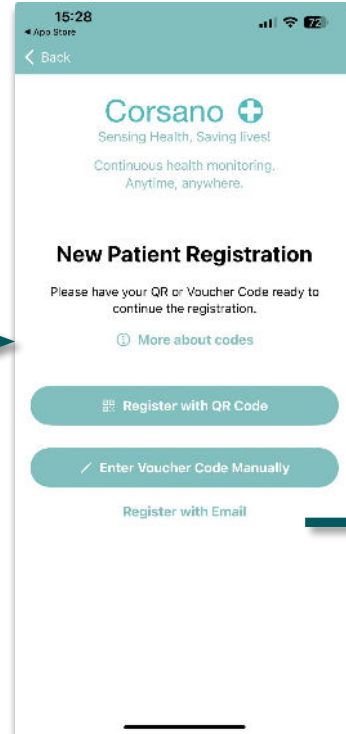


2) Select the option [Register with Voucher Code]

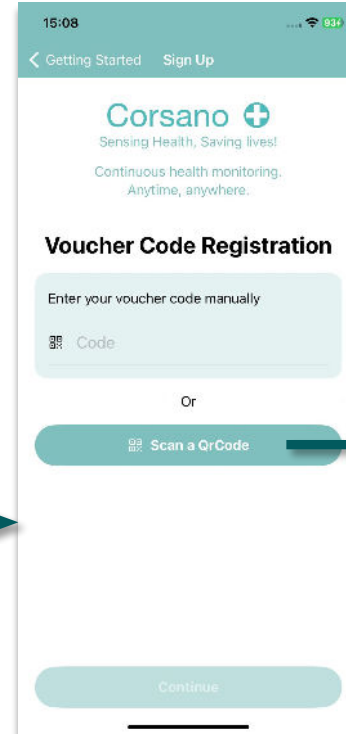

3) Click on [Scan a QR Code] to open the camera

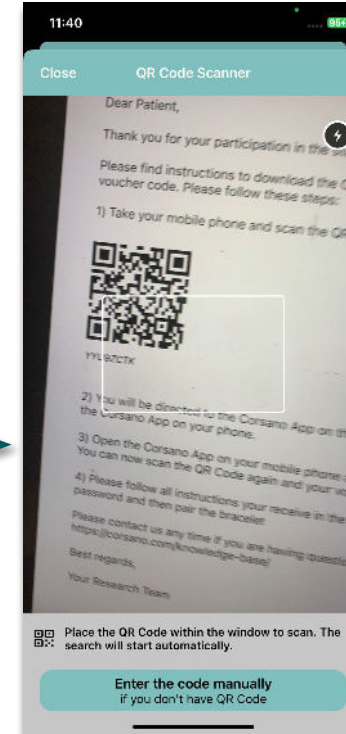

4) Scan the QR Code with the camera

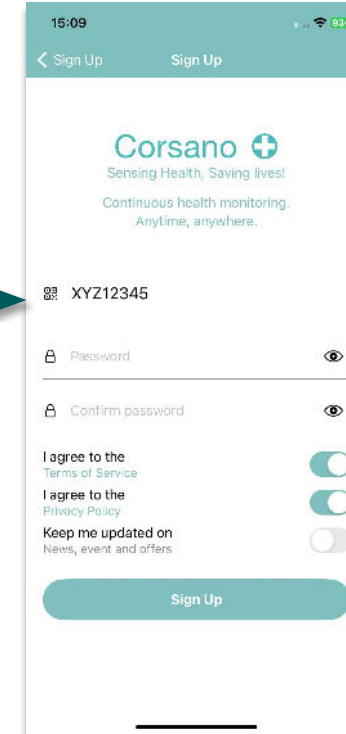


5) Voucher code detected. Set your own password to finish the registration process




Store password in safe place!


3) Access Existing Account

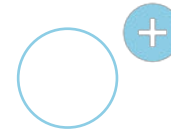


For patient if you already have an account

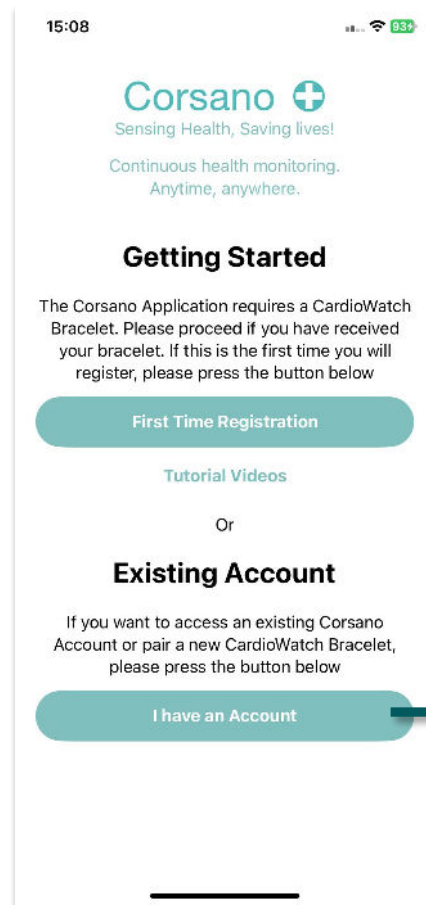

1) Press on [I have an Account]

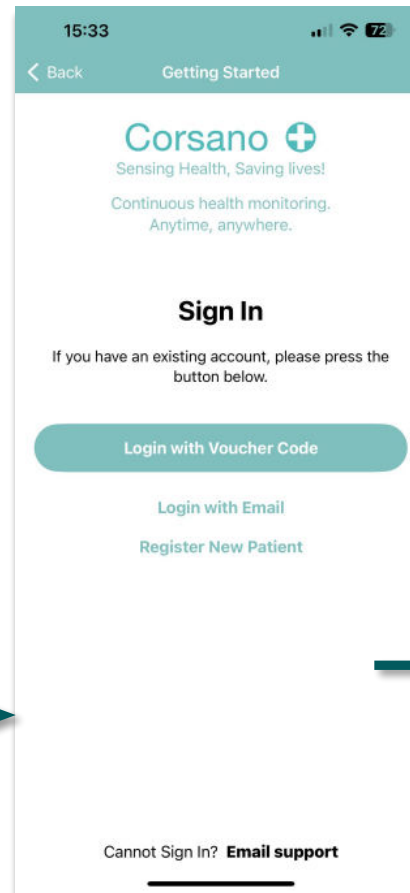

2) Select [Login with Voucher]

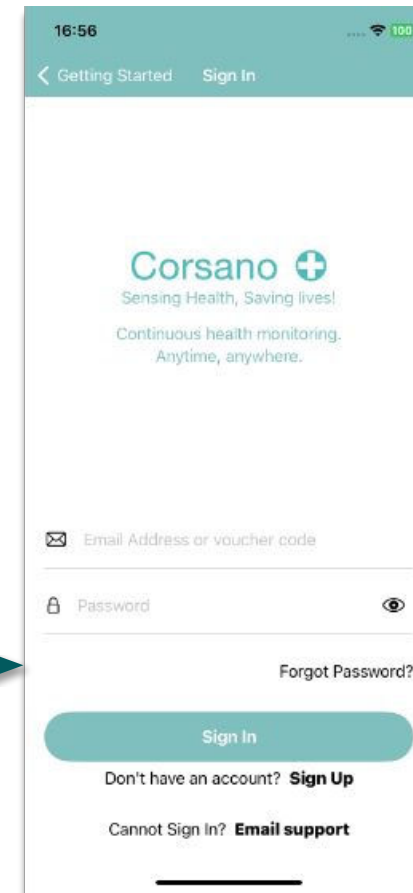

3) Enter your voucher code and your password