



1) Pair Bracelet - iOS

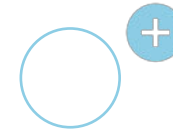

For patient if you pair the Corsano Bracelet for first time to your mobile device

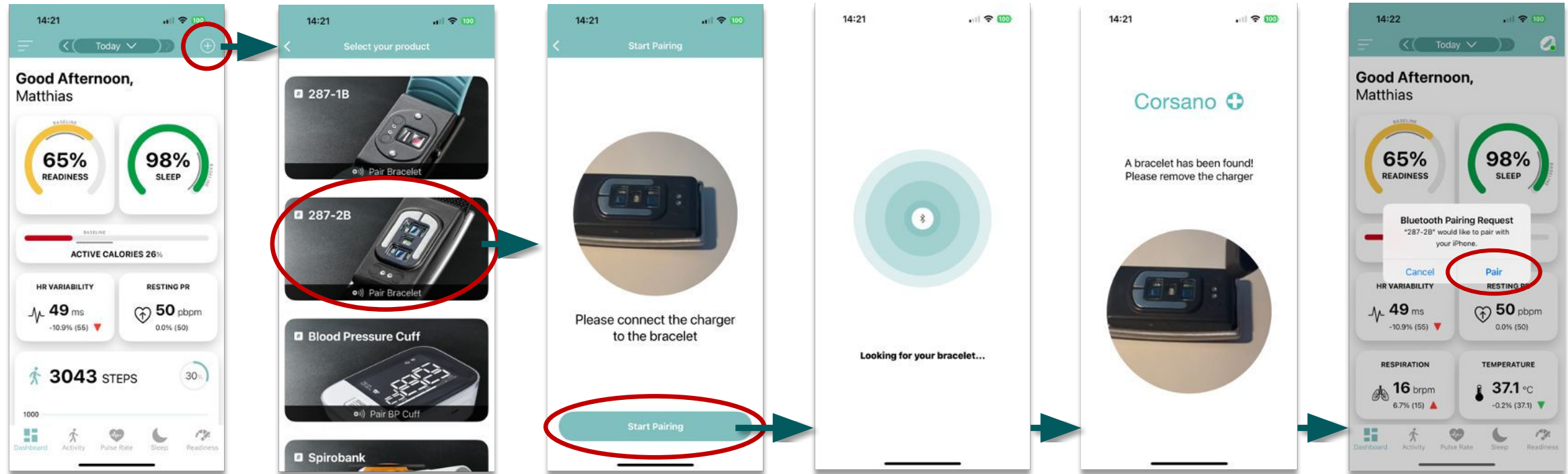

1) Press  right top corner to go to Settings

2) Select bracelet

3) Press [Start Pairing]

4) Wait for the end of the scanning process






5) Remove the charger from the bracelet to confirm the pairing

6) Secured bonding confirmation. Press [Pair]


1) Pair Bracelet - Android




For patient if you pair the Corsano Bracelet for first time to your mobile device

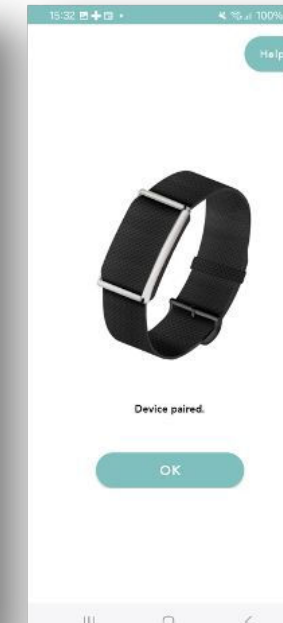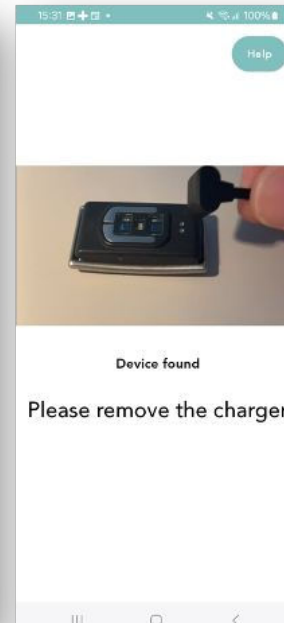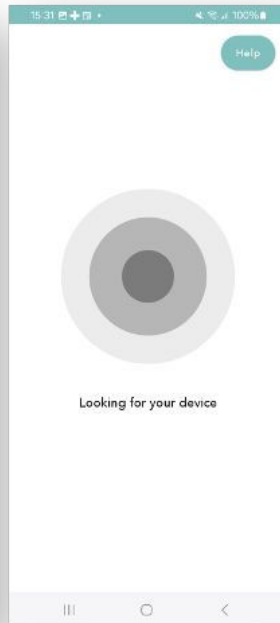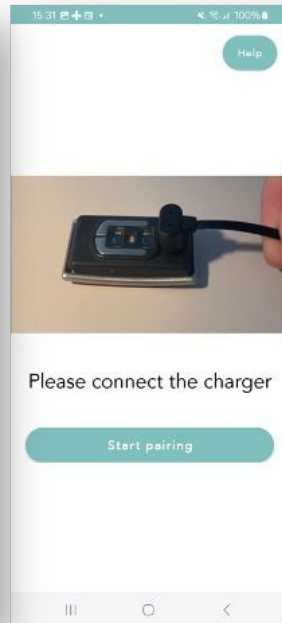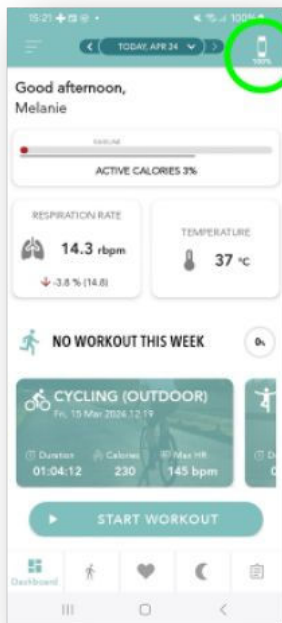


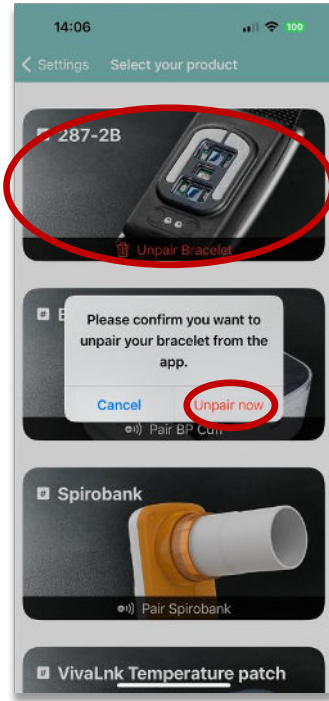
2) Unpair Bracelet - iOS

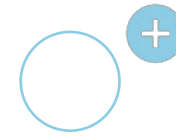


For patient if you previously paired a Corsano Bracelet to your mobile device

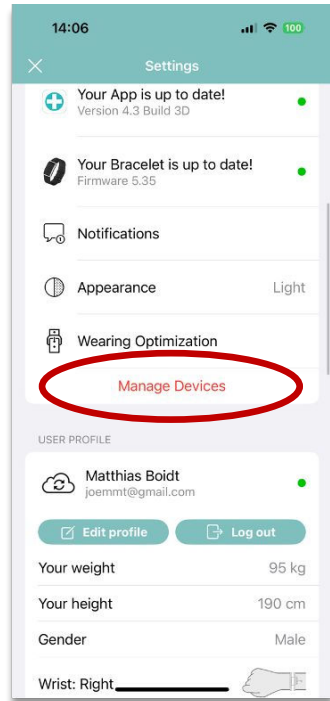


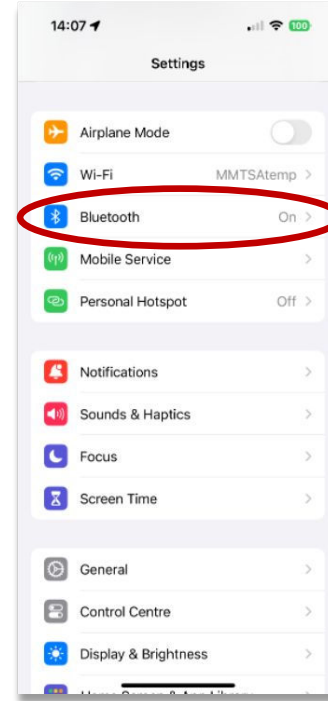
1) Press  top corner to go to Settings and select [Manage Devices]




2) Select bracelet and confirm when you receive the pop-up

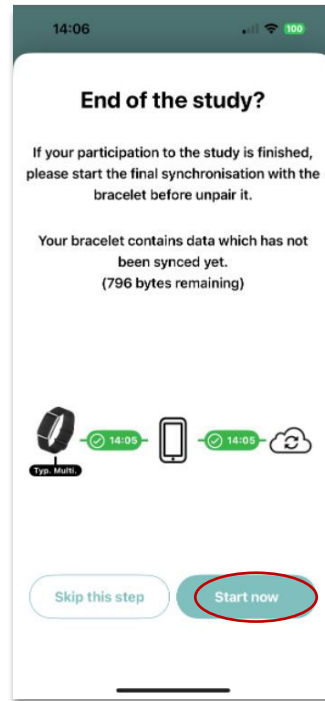


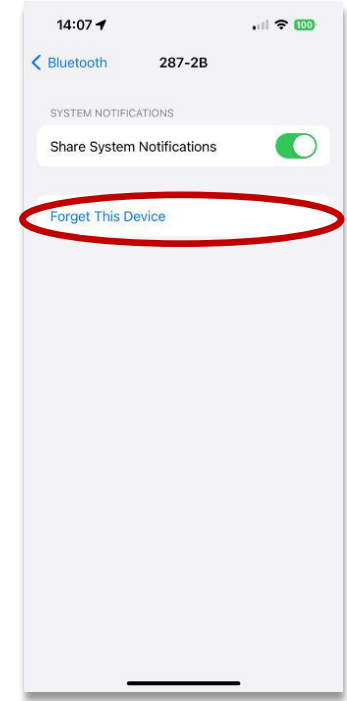
3) Press [Start now] and you will see  Wait until data is synced



4) You must remove the bracelet from mobile Bluetooth Settings. Go to 






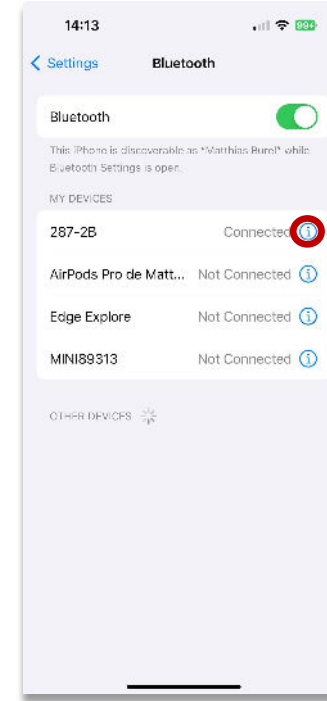


5) Select the 287-2B device in the iOS Bluetooth peripheral list. Click on the icon 



6) Click on [Forget This Device] to remove the secured bonding

2) Unpair Bracelet - Android

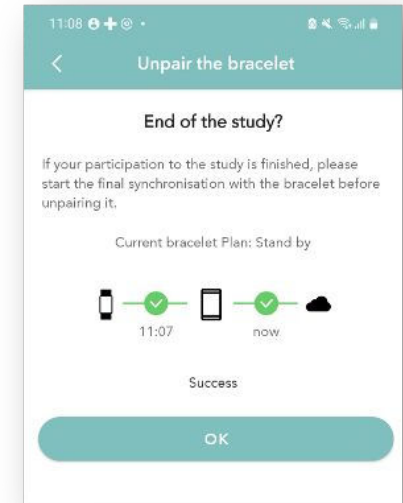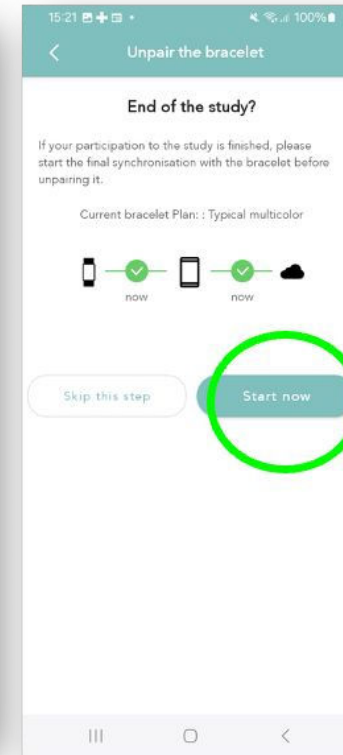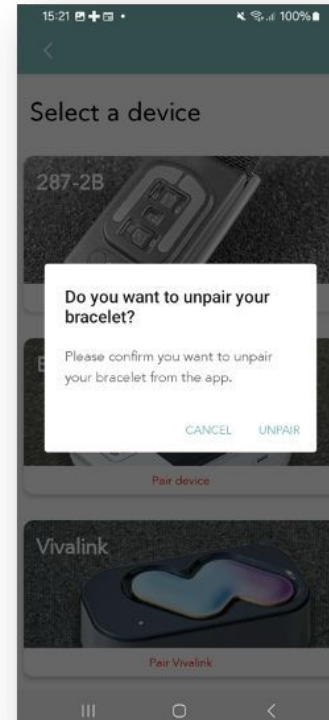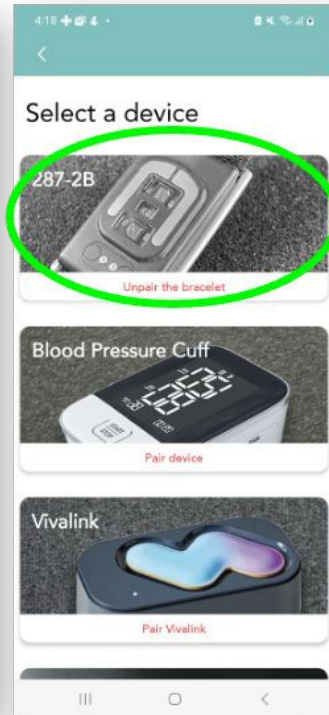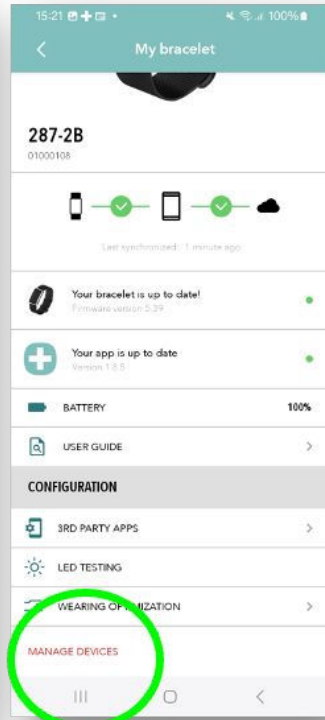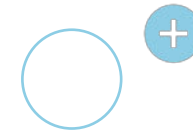

Bracelet lights are off
No monitoring

