


HCP Getting Started

1 Register & Prepare

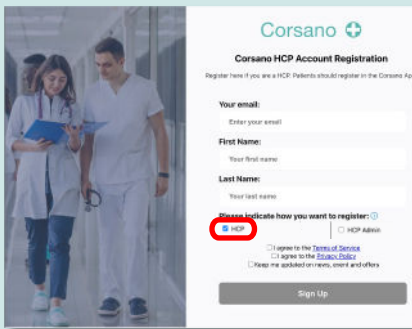

1) Go to <https://portal.corsano.com/> and select [Create New Account]. Select [HCP]. After completion, ask your HCP Admin to authorize



2) Install the HCP App, then select [First Time Registration]




3) Set Profile under Settings, enter HCP Code





4) Connect power cable or insert batteries to turn ON the BP Cuff




5) Press and hold the Start button of the BP Cuff Monitor for 5 sec, release when  blinks

2 Onboard New Patient

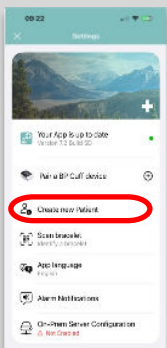

1) Press  left top corner and press [Create New Patient]

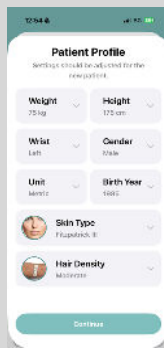

2) Update Patient Profile. Select closest Skin Color




3) Type Room Number or Patient ID to identify patient



4) Wearing Optimisation. Ask patient to sit or lie still




5) BP Cuff Measurement. Press [START] on monitor when prompted

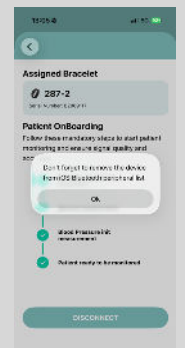


6) Disconnect to let Cisco take over syncing of data. Forget device

3 Wearing Tension & Position

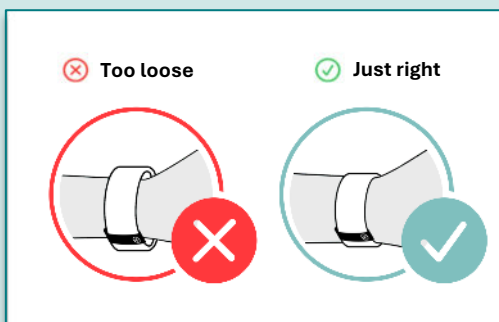

1) Tighten the strap so that it is secure against the skin, the green sensor light should not be visible. The Bracelet should be snug but comfortable

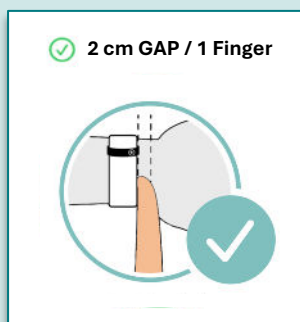

2) Position: Wear the Bracelet approx. 2 cm below the wrist

Corsano 

Tutorials

www.corsano.com/movies

Support

www.corsano.com/support

Follow Instructions
in the Corsano App

Corsano Health B.V.

The Hague Tech Office 1.07
Waldorpstraat 5
2521 CA Den Haag
The Netherlands